

# Disposable Rip Belts


**Instructions for use:**

The Disposable RIP belts come in 3 sizes:


- Small (S)
- Medium (M)
- Large (L)

For most patients RIP belts do not need to be adjusted if the correct size is chosen based on the patients abdomen circumference as listed in Chart 1 below. IF the abdomen circumference is not know the patients body mass index can be used for reference, refer to Chart 2 below. Note: The BMI reference is only valid for patients 18 years of age or older.

**Chart 1: Abdomen Circumference (AC)**

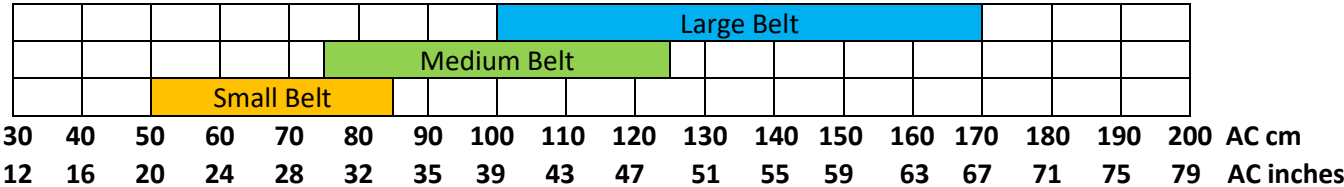

**Chart 2: Body Mass Index (BMI)**

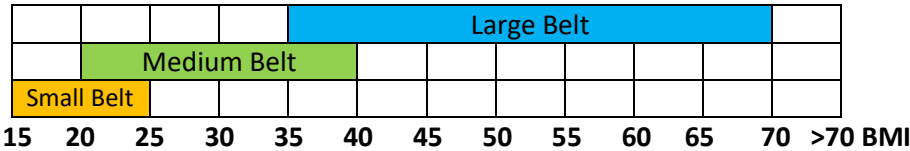

## Hook Up

For hook up, snap the belts on the snaps located on the cable and/or device to secure the belts around the patient’s abdomen and thorax as shown in the figures to the right.

For good signal quality, the belts should fit snugly without being uncomfortable tight.

